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Rev. Tarihi : 09.05.2019

Symbol Definitons

Caution, consult accompanying documents

Consult instructions for use

Non sterile

Do not reuse - Single Use

Catalog number

360 190 090 120

Example:

Brand Code
ACF

Cut Edge
19 mm

Cut Depth
90 mm

Thickness
1,20 mm

Batch code

Manufacturer

Manufacturing Date

Do not use if package is damaged

Quantity

INSTRUCTIONS FOR USE

Non-Sterile Surgical Saw Blade

Surgical Saw Blades are used by specialist surgeons, technicians in the field of orthopedic and traumatology for shortening time of cutting and revision of operational process of large bones (Femur, Tibia and Humerus) in orthopedic surgery. Blades can be used for the functional control by bioengineers before the operation.


Sterilization Recommendations

Saw blades are produced as non-sterile. The most effective and safe sterilization method for surgical saw blades is steam sterilization. So it should be unpacked and sterilized before using with recommended parameters in steam autoclave devices. Other sterilization methods are not recommended.

Parameters for Sterilizing

Sterilization Type	Minimum Temperature	Minimum Exposure	Dry Time
Pre-Vacuum	273° F (134° C)	4 Minutes	10-30 Minutes

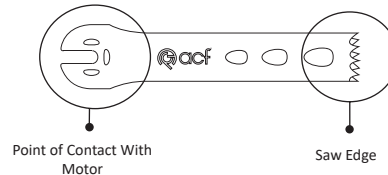Surgical saw blades are using with motor as follows:

WARNINGS

- W1. Saw blades are produced as non-sterile so it should be sterilized.
- W2. It is recommended to have a spare blade during operation.
- W3. While operating the surgical saw blade, the principles of aseptic technic should be followed after sterilizing it.
- W4. It should be used in the operating room environment where appropriate conditions are provided for orthopedic operations.
- W5. If the patient shows signs of irritation and / or sensitization, the operation should be stopped.
- W6. Re-using is not allowed. The operator is responsible for any risk that may arise from the saw blades used for the second time.
- W7. If the packages are damaged, return the saw blades to the manufacturer without using.
- W8. All staff (doctors, assistants, etc.) who will install and use the product should be familiar with surgical saw blades and the related operation.
- W9. Ensure that you have chosen an appropriate saw blade for your cutting guide.
- W10. Monitor the temperature of the saw blade periodically. Using irrigation while operating blades will help reduce the possibility of thermal necrosis.
- W11. Ensure that the saw blade is properly inserted and securely locked before activating the tool.
- W12. The blades must be able to move freely within the guide.
- W13. Forceful some other side rather than oscillation may cause the fracture of the saw blade.
- W14. A blade with a bent axe will wobble and should not be used.
- W15. Eye protection must be worn at all times.
- W16. If you notice any visible dirtiness, crack or trace on the saw blades, do not use and return them to to manufacturer.
- W17. Before the operation, remove the saw blade from the packaging, sterile at recommended conditions.
- W18. Ensure that the used blade is disposed according to the legal medical waste regulations referred to in international agreements.
- W19. Use care when handling sharps to prevent accidental cutting.
- W20. Do not exceed the manufacturer's recommended operating limits for the tool.

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ACF MEDİKAL ÜRÜNLER MAKİNA SAN. VE TİC. LTD. ŞTİ.